


Instructions for Use: Probe Covers

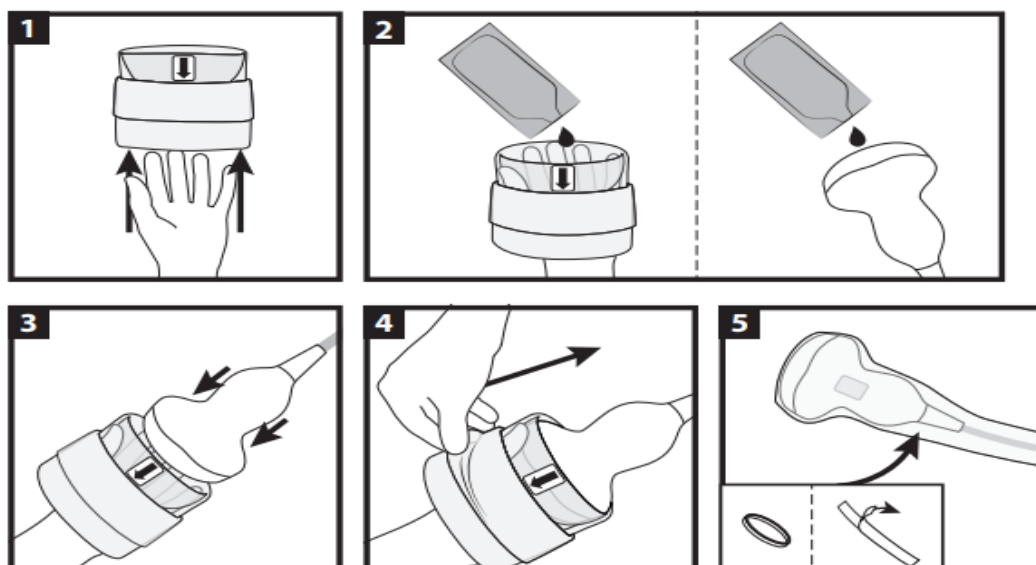

Recommended Storage Conditions:

General Warehousing - Store in a clean, dry, and well-ventilated area, away from direct sunlight, moisture, and extreme temperatures.

Intended Use:

The Non-Sterile Drapes/Covers for carts and general equipment offer protection against airborne contaminants during equipment transfer between rooms and times between use, by encapsulating carts and equipment by forming a barrier from the surrounding environment.

Intended User:

Intended to be used by Healthcare Professionals.














Caution: Single Use Devices Only: Reusing or reprocessing single use devices may cause damage to the device which in turn may cause unnecessary harm to the user and/or patient. Reuse or reprocessing is not recommended

Instructions for Use:

1. Read accompanying coupling gel instructions. (If Applicable)
2. Pull the probe cover over the probe until fully extended.
3. Fixate the probe cover with provided elastic bands. (If Applicable)
4. Do not use if the probe is damaged.

Notice:

Any serious incident that has occurred in relation to the device should be reported to Vital Care Industries, the governing body, and the Competent authority of the member state in which the incident occurred.

Symbol	Meaning
	Manufacturer
	Authorized representative in the European Community
	Consult Instructions for Use
	Batch Code
	CE Mark
	Catalogue Number
	Medical Device
	Do not reuse
	No polyvinyl chloride
	Date of Manufacture
	Country of Manufacture
	Importer
	Non-Sterile

